




1. Remove the healing components from the implants with the 1.25mmD Hex Tool.

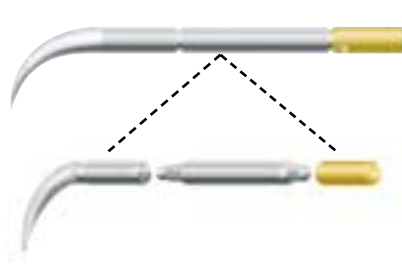

2. The Locator Core Tool consists of three pieces:  
1) Male Component Removal Tool  
2) Male Component Seating Tool  
3) Locator Abutment Driver



3. Insert the Locator Abutment into the implant and rotate into position using the Abutment Driver. Choose a cuff length based on the tissue depth. The cuff should extend 1mm above the tissue.



4. Insert a 1.25mmD Hex Tool into the back end of the Abutment Driver. Apply a minimum of 20 Ncm of torque with a calibrated Torque Wrench. As an alternative, use the Locator Torque Wrench Insert Driver with the Torque Wrench for tightening of the abutment.



5. Attach the Parallel Posts to the abutments to determine the degree of divergence. If the divergence is less than 10° use the white, pink or blue Locator Male. If the angle is greater than 10° use an Extended Range Male (green or red).



6a. Place the Locator Impression Coping onto the abutment. Inject light body impression material around the transfers and record a full-arch impression with standard body material.

Impression technique using Impression Coping



Reline technique using Processing Cap



Abutment Analog



6b. An optional technique is to use the patient's prosthesis in combination with the Cap with Black Processing Male for a reline impression.

7. Transfer technique: Insert the Abutment Analogs into the Impression Copings located within the impression material.

Reline technique: Attach the analog to the cap prior to inserting the assembly into impression.

8. The laboratory fabricates the working cast. Place the White Block-Out Spacer over the head of the Locator Abutment Analog. Attach the Cap with Black Processing Male. Process the denture following standard procedures.



9. Use the Locator Male Removal Tool (curved end) and wedge the hook against the bottom inside corner of the plastic male. Pull on the plastic male at an angle and remove it from the metal cap.



10. Use the Locator Male Seating Tool to firmly push the Locator Replacement Male into the empty metal cap located within the denture. The male component must sit flush with the rim of the metal cap.



11. The Locator Abutments should be retightened with a minimum of 20 Ncm of torque. Here the completed restoration is in place.

Note: This guide is for educational use only.  
See instructions for use for full instructions on the use of this product.

## Locator Abutments

Qty.	Tapered Screw-Vent® Implant System	
	Platform Diameter	Cuff Length
	TLOC3/0	3.5mm 0mm
	TLOC3/1	3.5mm 1mm
	TLOC3/2	3.5mm 2mm
	TLOC3/3	3.5mm 3mm
	TLOC3/4	3.5mm 4mm
	TLOC4/0	4.5mm 0mm
	TLOC4/1	4.5mm 1mm
	TLOC4/2	4.5mm 2mm
	TLOC4/3	4.5mm 3mm
	TLOC4/4	4.5mm 4mm

Qty.	AdVent® Implant System	
	Platform Diameter	Cuff Length
	AVLOC4/1	4.5mm 0.7mm
	AVLOC4/2	4.5mm 2mm
	AVLOC4/3	4.5mm 3mm
	AVLOC4/4	4.5mm 4mm
	AVLOC4/5	4.5mm 5mm
	AVLOC4/6	4.5mm 6mm

Qty.	SwissPlus® Implant System	
	Platform Diameter	Cuff Length
	OPLOC3/1	3.8mm 1mm
	OPLOC3/2	3.8mm 2mm
	OPLOC3/3	3.8mm 3mm
	OPLOC3/4	3.8mm 4mm
	OPLOC4/1	4.8mm 1mm
	OPLOC4/2	4.8mm 2mm
	OPLOC4/3	4.8mm 3mm
	OPLOC4/4	4.8mm 4mm

## Locator Abutments

Qty.	Spline® Implant System	
	Platform Diameter	Cuff Length
	SLOC3/1	3.25mm 1mm
	SLOC3/2	3.25mm 2mm
	SLOC3/3	3.25mm 3mm
	SLOC3/4	3.25mm 4mm
	SLOC3/5	3.25mm 5mm
	SLOC4/1	4.0mm 1mm
	SLOC4/2	4.0mm 2mm
	SLOC4/3	4.0mm 3mm
	SLOC4/4	4.0mm 4mm
	SLOC4/5	4.0mm 5mm
	SLOC5/1	5.0mm 0.86mm
	SLOC5/2	5.0mm 2mm
	SLOC5/3	5.0mm 3mm
	SLOC5/4	5.0mm 4mm
	SLOC5/5	5.0mm 5mm

## Abutment Analogs

Qty.	Description
	Body Diameter
	LOCAN/4 4.0mm (4 pack)
	LOCAN/5 5.0mm (4 pack) (for 4.8mmD SwissPlus and 5.0mmD Spline)

## Ancillary Components

Qty.	Description
	LOCMP2 Locator Male Processing Package (2 pack)
	LOCMP10 Locator Male Processing Package (10 pack)
	LOCIMP Locator Impression Coping (4 pack)
	LOCRMW Locator Replacement Male standard retention, white (4 pack)
	LOCRMP Locator Replacement Male light retention, pink (4 pack)
	LOCRMB Locator Replacement Male extra light retention, blue (4 pack)
	LOCRMG Locator Extended Range Replacement Male, green (4 pack)
	LOCRMRR Locator Extra Light Extended Range Rep. Male, red (4 pack)
	LOCPP Locator Parallel Post
	LOCCT 3-Piece Locator Core Tool (Male Removal Tool, Male Seating Tool and Abutment Driver)
	LOCTW Locator Torque Wrench Insert Driver
	TW30 Prosthetic Torque Wrench 30 Ncm
	TW20 Prosthetic Torque Wrench 20 Ncm
	TW1.25 Torque Wrench Insert Driver 1.25mmD
	TW1.25L Long Torque Wrench Insert Driver 1.25mmD

For a complete list of products and tooling, please refer to the current product catalog.

Customer Account \_\_\_\_\_

Ship To: Dr. Name \_\_\_\_\_

Phone ( ) \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_

State \_\_\_\_\_ Zip \_\_\_\_\_

Order Date \_\_\_\_\_

Ordered By \_\_\_\_\_

### Ship Via Federal Express

2-Day  Overnight (5:30 pm)  P1 (10:30 am)

Must Arrive By \_\_\_\_\_

### Payment Options

Invoice  COD  Visa  Mastercard  AmEx

Credit Card # \_\_\_\_\_

Exp. Date \_\_\_\_\_

Signature \_\_\_\_\_

### For questions or to fax orders:

In the U.S. 800 854 7019  
Fax 888 225 2483

Outside the U.S. +1 760 929 4300  
Fax +1 760 431 7811

Australia +61 2 4297 4182  
or 1800 241 916  
Fax +61 2 429 74190

Canada +1 800 265 0968  
or 1 905 567 2073  
Fax +1 905 567 2076

France +33 (0)1 45 12 35 35  
Fax +33 (0)1 45 60 04 88

Germany +49 (0)761 4584 722/723  
Fax +49 (0)761 4584 729

Israel +972 (0)3 6124242  
Fax +972 (0)3 6124243

Spain +34 93 846 05 43  
Fax +34 93 845 43 25

www.zimmerdental.com