


Facial-Lingual Pulls

Visual Appearance: V-shaped void, trough-like.

Result: Failure to capture complete and accurate dentition.

CAUSE	SOLUTION
Exceeding the working time.	Follow manufacturer's working time specifications. Choose material with longer working time.
Tray movement or repositioning after seating.	Do not move tray after seating.
Insufficient amount of impression material used.	Use more material to create back flow effect.
Impression tray does not support flow of impression material.	Use lingual stops. Use an impression tray that supports the flow of the material.

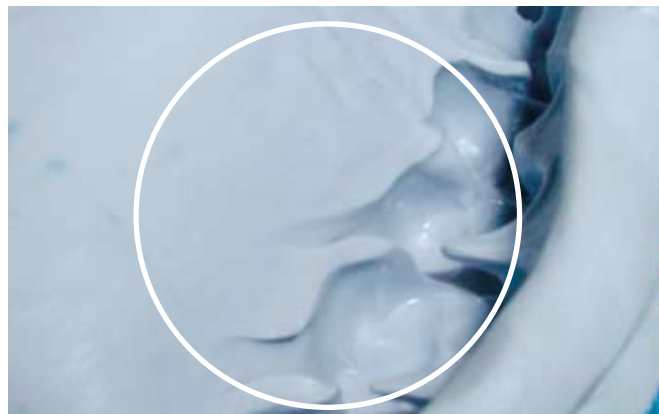

Lingual Pulls